

wartie®

Wart & Plantar Wart Remover

What is Wartie and what it is used for?

Wartie® is an easy to use 'freeze' treatment for warts and plantar warts (verruca), similar to the cryotherapy method used by doctors.

Warts are small growths of the skin caused by the human papilloma virus (HPV).

Warts often develop on the fingers, or on the back of the hands, knees and elbows. You can recognise the common wart by the rough 'cauliflower-like' appearance of the surface. The wart virus, which is very contagious, can be transmitted from person to person or from one body location to another by direct contact with an infected area.

Plantar warts (verruca) are similar to warts but occur only on the sole of the foot or underneath the toes. Generally painful, they often look like a small white ring of skin with black dots in the centre. They are also contagious and can be transmitted to another person by contact with a wet surface such as a floor.

Only use Wartie® if you are sure the skin condition is a wart (common or plantar).

If you are not sure, consult your general practitioner. If you treat a skin condition as a wart with Wartie® and it is not a wart, you delay appropriate treatment. This delay may lead to a more serious skin condition or disease.

How does Wartie® work?

Wartie® precisely freezes the wart or plantar wart to the core, due to its unique precision tip. The rounded shape of the Wartie® tip allows accurate application limited to the wart or plantar wart, thereby avoiding the surrounding healthy skin, resulting in a more pain-free application.



Precision tip

- Targets only the wart or plantar wart (verruca)
- Outstanding freezing performance
- No direct contact between gas and skin

When using cryotherapy devices for wart or plantar wart removal, the area treated may temporarily turn white. **After the frozen area thaws, it will then turn red. A blister may form underneath the treated wart or plantar wart as a result of the freezing process. The wart or plantar wart will gradually disappear or fall off over the next 10 to 14 days, revealing healthy new skin underneath.** One application should be sufficient to successfully remove a wart or plantar wart. Should a second treatment be necessary please wait 14 days between applications. **In some cases, it may help to soak the wart or plantar wart in warm water for a few minutes before using Wartie®, in order to help penetrate callused skin that has formed on top of it.**

Why Wartie?

Wartie® employs a patented technology to provide a fast and effective treatment for warts and plantar warts.


Wartie is Fast

Wartie® does not require any assembly, thereby providing a quick and easy treatment for warts and plantar warts. Wartie® is ready to use straight out of the packaging without tricky assembly of sponges and small pieces. After only 5 seconds preparation the product can be applied on the wart or plantar wart.

Wartie is Precise

The unique design of Wartie® has the advantage of making the treatment of a wart or plantar wart faster while avoiding direct contact between the skin and the liquid gas inside the aerosol. Because Wartie® delivers the cold through a metal tip, the freezing power can be controlled by the amount of pressure applied to the can during application. More pressure means more of the metal tip touching the wart, resulting in a larger freezing area. With only light pressure, the contact is minimised, thereby being able to treat smaller warts or plantar warts.


Freezing precision

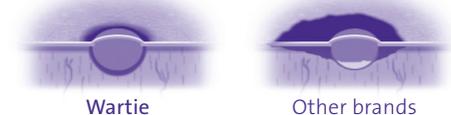

Wartie is Effective

The freezing power of Wartie® has been compared to a number of other cryotherapy products.

Outstanding freezing performance

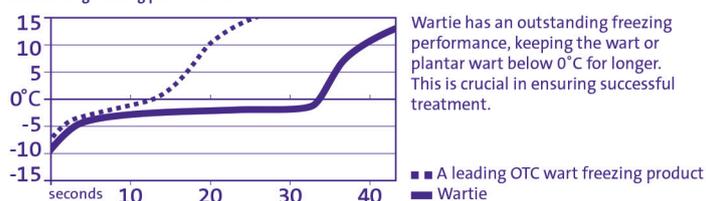

How to use Wartie?

Use only as directed.

<p>1 Place the can on a table and hold the collar</p>	<p>2 Leave the can on the table. Turn the cone to 1 and press it down for 3 seconds (DO NOT EXCEED 3 SECONDS). You hear a hiss.</p>	<p>3 The cone automatically returns to "0" and the metal tip starts cooling. Wait for 2 seconds.</p>	<p>4 Apply the tip precisely on the wart or plantar wart to freeze it</p>

Outstanding freezing performance

- Revolutionary ease of use
- Precisely targets the wart or plantar wart (verruca)
- Suitable for children aged 4+

1. Remove the cap and place the can on a table or flat surface holding it by the collar.
2. **Keeping the can in vertical position (do not tilt):** Firmly hold the collar between your thumb and index finger, twist the cone in counter clockwise direction with your other hand and firmly push down the cone for 3 seconds without touching the metal tip. You will now hear the activation sound (hiss).
3. **Be sure the cone twists back to zero automatically to lock the system before applying the product.** Wait for 2 seconds while the metal tip freezes.
4. Precisely apply the frozen metal tip on the wart (hand) for 20 seconds or on the plantar wart (foot) for 40 seconds to freeze it. It is recommended to use a timer.

When you apply the frozen Wartie® tip on warts or plantar warts, a slight aching sensation may occur as a result of the freezing process. This is normal and should disappear within a few hours. One application should be sufficient to successfully remove a wart or plantar wart. Should a second treatment be necessary please wait 14 days between applications.

Allow the metal tip to reach room temperature after each application before reactivating for use on other warts or plantar warts - this takes approximately 5 minutes. Wash hands (except treated area) immediately after use. Washing the treated area immediately may be uncomfortable since you may still be experiencing some aching/stinging. The treated area may be washed after the aching/stinging subsides.

Follow-up treatment

- Do not pick or scratch the treated area as this may result in infection.
- Protect blisters, if necessary, with a gauze or sterile adhesive bandage.
- Do not puncture blisters. Puncturing blisters will be painful and may result in infection.
- Avoid sun exposure or wear sunscreen (SPF 50) and minimise trauma to treated sites until completely healed (i.e wart/plantar wart has fallen off and colour has returned to normal).

Cautions

- Do not use Wartie® if:
 - the cone is stuck to position "1" after activation, and/or,
 - you do not hear the "hissing" sound during activation (release of aerosol), and/or,
 - the "hissing" sound continues after activation, and/or,
 - the tip does not become cold after activation, an "icy surface" should become visible on the metal tip,
- In case one or more of the above mentioned conditions are not fulfilled, the device may not be functioning properly.
- Failure of the device may lead to injury or insufficient/incomplete treatment.
- Do not use Wartie®
 - on children under the age of 4 years.
 - anywhere in, on or near your face, eyes, nose, mouth, ears, armpits, breasts, bottom or genital area.
 - on birthmarks, moles, warts with hairs growing from them, or any other spots.
 - on areas where your skin is sensitive, inflamed, damaged, cut, grazed, diseased or itchy.
 - if you cannot clearly see the wart you are treating.
 - if you are diabetic or suffer from poor circulation.
 - if you are pregnant or breastfeeding.
 - together with any other method of wart removal because it is uncertain how the combination may affect your skin. Combination usage may result in serious burns and permanent scarring of the skin.
- The metal tip is coated with nickel, which may cause an allergic reaction in some individuals.
- If you are not sure about the condition of the unwanted wart or plantar wart, consult your general practitioner.

Warnings

Use only as directed. Careless use can cause freezing too deeply, causing permanent damage to the uppermost layer of the skin on fingers and toes. This may result in permanent scarring or nerve damage.

If you mistakenly apply Wartie® on conditions that are not warts, it may cause serious burns and permanent scarring of the skin.

Keep out of reach of children.

Aerosol may burst if heated. **Extremely Flammable.** Do not pierce or burn, even after use. Keep away from heat/sparks/open flames/hot surfaces - no smoking. Do not expose to an open flame or other ignition source. Do not expose to temperatures exceeding 50°C. Protect from sunlight.

Avoid Contact with Eyes. Contact with eyes may cause blindness. Do not rub or touch eyes with hands if Wartie® has been applied to them. If product gets into the eyes, flush with water for 15 minutes and seek medical help immediately.

Do not Swallow or Inhale the product. For External Use Only. Content is toxic and may cause serious internal damage. If swallowed, inhaled or sprayed on body parts other than warts, seek medical help or contact a Poison Information Centre right away.

For safety reasons, Wartie® must always be applied by an adult. Do not apply excessive pressure when pushing the applicator on to the wart or plantar wart. The applicator should be applied for a maximum of 20 sec (wart) / 40 sec (plantar wart). If being used by more than one person, clean the applicator with alcohol on a small piece of cloth after each application. If there are multiple warts or plantar warts close to each other, treat only one at a time with a two-week period between treatments, for a maximum of three times in total.

Side effects

- As with all cryotherapy devices for wart removal, the skin may temporarily turn white then red.
- A slight aching, itching and/or stinging feeling will occur as a result of the freezing. After the applicator is removed, colour will return and the aching, itching, stinging feeling will decrease, disappearing within a few hours for common warts and within 24 hours for plantar warts. If discomfort or an aching/stinging feeling lasts beyond this time, talk to your general practitioner.
- Black dots may appear in the centre of the treated wart after a few days.
- Blisters resulting from the treatment of a larger or more persistent wart or plantar wart can be more uncomfortable.
- After the more persistent wart or plantar wart has come off, de-pigmentation or a small scar may occur.
- Using cryotherapy products for freezing warts or plantar warts may result in minor burn wounds due to the low freezing temperature. If more discomfort is experienced than would be expected, consult your general practitioner.

Wartie contains:

Dimethyl ether

The product can be used until the expiration date indicated on the base of the aerosol can.

Store below 25°C. Do not refrigerate or freeze.

For more information and an instructional video please visit www.wartie.com.au

Manufacturer:

YouMedical
YouMedical B.V.,
Rijnsburgstraat 9-11,
1059 AT Amsterdam,
The Netherlands.

Distributor:

Key Pharmaceuticals Pty Ltd
12 Lyonpark Rd, Macquarie Park NSW 2113
Toll Free Helpline: 1800 653 373
www.keypharm.com.au

Distributed in New Zealand by:

Wilson Consumer Products Ltd.
26-28 Hobson St, Auckland Central,
New Zealand
Toll Free Helpline: 0800 246 456